

# EndoPilot

## Step-by-Step Setup Guide



# 1

Make sure that you have all of the required EndoPilot components.



## Components:

- |   |                               |   |                                 |
|---|-------------------------------|---|---------------------------------|
| ① | EndoPilot control unit        | ⑥ | Lip clip                        |
| ② | Wireless remote foot switch   | ⑦ | File clip                       |
| ③ | Handpiece motor's (blue plug) | ⑧ | File clip connector cable       |
| ④ | Contra-angle                  | ⑨ | Apex locator cable (green plug) |
| ⑤ | File clip holder              |   |                                 |

# 2

Plug in the green plug of the apex locator cable (9) to the color-coded port on the right side of the EndoPilot. Then plug the file clip connector cable (8) into the end of the apex plug.



3

Plug the black file clip (7) into the other end of the connector cable (8) and then plug the lip clip (6) into the remaining end of the apex cord (9).



4

Plug the EndoPilot Handpiece motor's (3) blue plug into the blue receptacle on the right side of the EndoPilot, then connect the contra-angle (4) to the handpiece.



5


Turn on the EndoPilot using the power button located on the rear of the device. Plug in your EndoPilot and if battery indicator on the touchscreen reads less than 80%, plug in the charging cable and wait until indicator is over 80%.





6

Calibrate the Motor – from the Home Screen, Select Treatment, then on the treatment screen, select the options button (it looks like a gear in the upper right corner). Then select “Calibrate”. For optimal results the EndoPilot should be calibrated daily.

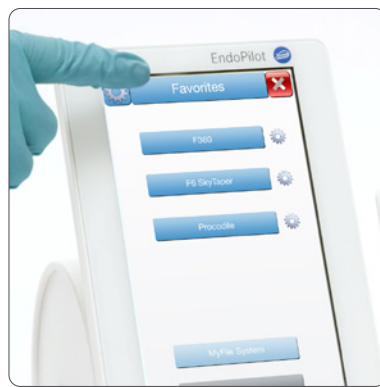

7

X-out – Select File System and start your treatment



For further information  
on the EndoPilot

